# RVENT

CATHETER MOUNT INSTRUCTION FOR USE **EN**

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## CATHETER MOUNT INSTRUCTION FOR USE

### INTENDED USE:

Catheter Mount is an airway connector for conduction of respiratory gases between the breathing system and the patient interface (e.g. endotracheal tube, mask) of a mechanically ventilated patient.

### WARNINGS:

- The product is manufactured for single use only.
- The medical device is only to be used for the purpose specified under "Intended Use" section.
- The medical device must not be reused or reprocessed.
- To avoid contamination and soiling, keep the medical device packaged until ready to be used. Do not use the medical device if the packaging is damaged.
- Do not modify the medical device. Modification may damage or impair the proper functioning of the device which may lead to patient injury.
- Perform a self-test of the basic device including a leakage test after the breathing circuit (hose, filter/HME, etc.) has been completely installed and before use on the patient.
- If it includes a protective cap on the patient side connector, remove the protective cap immediately before use on the patient.
- The gas sampling port must only be used for monitoring breathing gas and anesthesia gases.
- The port for suctioning and bronchoscopy must only be used for suction and bronchoscopy. When the port for suctioning or bronchoscopy is used, it must only be opened immediately prior to introduction of suction catheter or bronchoscope due to the risk of high leakage and insufficient patient ventilation. For the same reason, it must be closed immediately after removal of suction catheter or bronchoscope. Check all the connections are secure.
- The introduction of gases or liquids via the gas sampling port or the port for suctioning and bronchoscopy can cause serious injury to the patient.
- Regularly check the inside of the medical device for liquid or secretion accumulation. Liquids or secretion can lead to increased resistance or enter the patient's airways.
- If includes; a Luer Lock connector must only be used for gas monitoring. Any other use of the Luer Lock connector may endanger the patient. When not in use, the Luer Lock connector must be sealed with its cap.

### RISK OF FIRE:

In combination with oxygen or nitrous oxide, ignition sources such as electrosurgery and laser surgery devices can cause fires. To protect patient and users, prevent leakages from hoses carrying oxygen or nitrous oxide. Before beginning electrosurgery or laser surgery flush the vicinity of gas-carrying parts (endotracheal tube, mask, Y-piece, hoses, filter, and breathing bag) sufficiently with air (<25 % O<sub>2</sub>) flush beneath the surgical drapes as well. In order not to damage the breathing circuit, keep a distance of at least 200 mm (7.9 in) between hoses carrying oxygen or nitrous oxide and a possible ignition source (e.g., electrosurgery or laser surgery devices).

### If the product is packaged sterile;

- The device is sterilized with Ethylene Oxide. Sterile conditions are guaranteed only if the package is not damaged.

**STERILE EO**

- Do not re-sterilize. Re-sterilizing damages the product integrity.

### INSTRUCTION FOR USE:

The medical device must only be used on the patient side. Connect the device-side connector of the catheter mount to the patient connection port of the breathing circuit. If a filter or a heat and moisture exchanger is used, insert it between the airway connector and the breathing circuit.

### INDICATIONS:

The catheter mount is a flexible adapter that provides the connection of the endotracheal tube, face mask with the breathing circuit during treatment.

**CONTRAINDICATIONS:**

The device is used only as directed. Improper use of it will harm the patient.

**TIME OF USAGE:**

The device is intended for single use only. It is used during the application period

**TARGET PATIENT GROUP:**

Adult, pediatric and neonatal patients

**SHELF LIFE:**

For all of the Catheter Mounts, Shelf Life is 5 years.

**USERS:**

Users have the necessary expertise, training, and knowledge to use or install the medical device.

**DISPOSAL METHOD:**

Local regulations and/or hospital waste management procedures of the relevant country should be followed when disposing of the waste products.

**PERFORMANCE DATA:**

The performance data conforms to the relevant sections of the ISO 5367 standard. All breathing tube types (corrugated, smoothbore and extendible) and patients groups (adult, pediatric and neonatal) tested for resistance to flow and compliance according to relevant sections of ISO 5367 standard.

<b>Resistance to Flow at 30L/min:</b>	<0.06hPa//l/min
<b>Resistance to Flow at 15L/min:</b>	<0.12hPa//l/min
<b>Resistance to Flow at 2.5L/min:</b>	<0.74hPa//l/min
<b>Leakage at 60mbar:</b>	<b>For Adult:</b> <70ml/min
	<b>For Pediatric:</b> <40ml/min
	<b>For Neonatal:</b> <30ml/min
<b>Compliance at 60 hPa:</b>	<b>For Adult:</b> <5ml/hPa
	<b>For Pediatric:</b> <4ml/hPa
	<b>For Neonatal:</b> <1.5ml/hPa

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