RVENT

BVM RESUSCITATOR INSTRUCTION FOR USE **EN**

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BVM RESUSCITATOR INSTRUCTION FOR USE

INTENDED USE:

This product is intended for doing cardiopulmonary resuscitation and temporary ventilate in hospital, transport, emergency and post hospital care.

WARNINGS:

- Intended for use by qualified personnel trained in advanced pulmonary ventilation and/or ACLS techniques.
- Proficiency in the use of this product must be demonstrated prior to use on a patient.
- Clear patient's airways before using manual resuscitator.
- Always check if the resuscitator functions properly.
- Check if the valve functions properly. Check if the patient is being ventilated by observing the rise and fall of the chest. Color of the lips and face during resuscitation .
- Always make functional test of the resuscitator after unpacking and assembly.
- Do not use the resuscitator in toxic or hazardous atmospheres.
- Oil or grease should not be used in close proximity if with oxygen equipment – since this may cause fire.
- Do not smoke or exposed on flame when oxygen is in use this can cause fire.
- The correct ventilation frequency may vary. Please follow the current ventilation frequency recommended by national or international guidelines.
- If the patient vomits during mask ventilation, immediately clear the patient's airways of vomitus, then freely compress the bag a few times before resuming ventilation procedure.
- **Adult version:** The pressure limiting valve is open at 60 cmH₂O.
Child and infant version: The pressure limiting valve is open at 40 cmH₂O.
- If higher pressures are required, press and spin the button while squeezing the bag, the pressure limiting valve is overridden.
- Administer oxygen according to medical indications.

INSTRUCTION FOR USE:

Prior to use, read these instructions, cautions and warnings completely.

Assemble the manual resuscitator: Fully extend the bag of resuscitator. Prior to using the resuscitator, and Check the valve function by squeezing the resuscitator. Connect non- rebreathing valve into outlet of mask. If resuscitating with high oxygen concentration, fully extend the oxygen tube, attach oxygen nozzle adapter to proper oxygen source. Connect reservoir bag, set oxygen flow but do not exceed 15 LPM or comply with the requirement of a physician.

Open mouth, clear airways of all foreign matter and fluids. The use of an Emergency Aspirator is recommended. Tilt head fully backwards and push the jaw upwards with neck stretched to open the airways. To augment ventilation it may be beneficial to insert an artificial airway. Take precautions since it should not push the tongue back and thus obstruct the throat. Hold mask tightly to victim's face, covering mouth and nose, tilt head fully backwards, hold the mask hand lifting jaw forward. Squeeze the bag appropriately and observe the chest if it expand. Release pressure from the bag gradually and allow the chest to deflate. Do this for 12-20 times per minute, or 30 times on infants.

Note: If continued resistance to insufflation is experienced, check for airways obstruction or check if the head is tilted. If adequate ventilation is not achieved with the resuscitator, immediately revert to air ventilation (mouth-to-mouth, or mouth-to-nose).

INDICATIONS:

The product is intended to provide lung ventilation to individuals whose breathing is inadequate.

CONTRAINDICATIONS:

- Tension pneumothorax or mediastinal emphysema without decompression and drainage.
- Moderate or more serious hemoptysis.
- Serious Pulmonary cyst or pulmonary bullous.
- Hypovolemic shock without supplement the blood volume. Acute myocardial infarction.
- Suffocating respiratory failure caused by severe aspiration.

SHELF LIFE: For BVM Resuscitators, Shelf Life is 3 years.

TARGET PATIENT GROUP:

Adult, pediatric and neonatal patients.

USERS:

Users have the necessary expertise, training, and knowledge to use or install the medical device.

CLEANING AND DISINFECTING:

This product is intended for single-patient use. DO NOT attempt to wash or disinfect. DO NOT sterilize or use on more than one patient. After the recommended period of use, discard the product according to normal procedures.

DISPOSAL METHOD:

Local regulations and/or hospital waste management procedures of the relevant country should be followed when disposing of the used products in order to avoid possible contamination.

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