
RVENT

- CLOSED SUCTION SYSTEM INSTRUCTION FOR USE** (EN)
- تعليمات استخدام نظام شفط مغلق (AR)
- NÁVOD K POUŽITÍ UZAVŘENÝ ODSÁVACÍ SYSTÉM** (CS)
- BRUGERVEJLEDNING TIL LUKKET SUGESYSTEM** (DA)
- GESCHLOSSENES ABSAUGSYSTEM GEBRAUCHSANWEISUNG** (DE)
- MANUAL DE USUARIO DEL SISTEMA DE ASPIRACIÓN CERRADO** (ES)
- MODE D'EMPLOI DU SYSTÈME CLOS D'ASPIRATION** (FR)
- ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ ΚΛΕΙΣΤΟ ΣΥΣΤΗΜΑ ΑΝΑΡΡΟΦΗΣΗΣ** (GR)
- ZÁRT SZÍVÓRENDSZER HASZNÁLATI UTASÍTÁS** (HU)
- SISTEMA DI ASPIRAZIONE CHIUSO ISTRUZIONI PER L'USO** (IT)
- ЖАБЫҚ СОРУ ЖҮЙЕСІ ПАЙДАЛАНУ ЖӨНІНДЕГІ НҰСҚАУЛЫҚ** (KAZ)
- GEbruikersHANDLEIDING GESLOTEN AFSYSTEM** (NL)
- ZAMKNIĘTY SYSTEM DO ODSYSANIA INSTRUKCJA UŻYTKOWANIA** (PL)
- SONDA DE ASPIRAÇÃO DE SISTEMA FECHADO INSTRUÇÕES DE USO** (PT)
- ЗАКРЫТАЯ АСПИРАЦИОННАЯ СИСТЕМА ИНСТРУКЦИЯ ПО ПРИМЕНЕНИЮ** (RU)
- ZATVORENI SISTEM ZA ENDOTRAHEALNU SUKCIJU UPUTSTVO ZA UPOTREBU** (SR)
- KAPALI EMME SİSTEMİ KULLANMA KILAVUZU** (TR)

- 1** CLOSED SUCTION SYSTEM INSTRUCTION FOR USE
- 5** تعليمات استخدام نظام شفط مغلق
- 9** NÁVOD K POUŽITÍ UZAVŘENÝ ODSÁVACÍ SYSTÉM
- 13** BRUGERVEJLEDNING TIL LUKKET SUGESYSTEM
- 17** GESCHLOSSENES ABSAUGSYSTEM GEBRAUCHSANWEISUNG
- 22** MANUAL DE USUARIO DEL SISTEMA DE ASPIRACIÓN CERRADO
- 27** MODE D'EMPLOI DU SYSTÈME CLOS D'ASPIRATION
- 32** ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ ΚΛΕΙΣΤΟ ΣΥΣΤΗΜΑ ΑΝΑΡΡΟΦΗΣΗΣ
- 37** ZÁRT SZÍVÓRENDSZER HASZNÁLATI UTASÍTÁS
- 41** SISTEMA DI ASPIRAZIONE CHIUSO ISTRUZIONI PER L'USO
- 45** ЖАБЫҚ СОРУ ЖҮЙЕСІ ПАЙДАЛАНУ ЖӨНІНДЕГІ НҰСҚАУЛЫҚ
- 49** GEBRUIKERSHANDLEIDING GESLOTEN AFSYSTEM
- 54** ZAMKNIĘTY SYSTEM DO ODSYSANIA INSTRUKCJA UŻYTKOWANIA
- 58** SONDA DE ASPIRAÇÃO DE SISTEMA FECHADO INSTRUÇÕES DE USO
- 62** ЗАКРЫТАЯ АСПИРАЦИОННАЯ СИСТЕМА ИНСТРУКЦИЯ ПО ПРИМЕНЕНИЮ
- 67** ZATVORENI SISTEM ZA ENDOTRAHEALNU SUKCIJU UPUTSTVO ZA UPOTREBU
- 71** KAPALI EMME SİSTEMİ KULLANMA KILAVUZU

CLOSED SUCTION SYSTEM INSTRUCTION FOR USE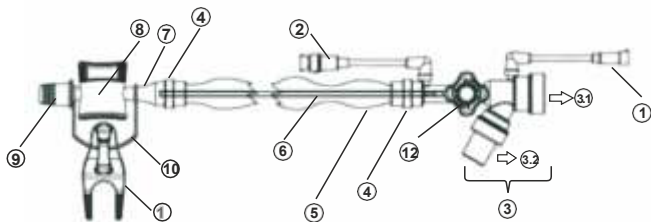

PRODUCT DIAGRAM:

- 1- MDI port
- 2- Washing port with reflow device with attached cap
- 3- Double elbow connector swivel (3.1. Adapter tube (elbow) to connect with ET or Tracheostomy tube and 3.2. Adapter tube (elbow) to connect with breathing system or ventilator
- 4- Connector for catheter protective sleeve
- 5- Catheter protective sleeve
- 6- Graduated suction catheter
- 7- Connector with or flow control valve
- 8- Suction or flow control valve with attached cap
- 9- Coupling cone to suction system with attached cap
- 10- Strap on safety cap
- 11- Disconnection wedge
- 12- Isolation valve*

*Isolation valve is only applicable in 72HRS model series.

INTENDED USE:

For clinical closed suction aspiration. The product is mainly used for respirators and patients undergoing endotracheal intubation to remove secretions and sputum from the patient's airway.

WARNINGS:

- Do not use more than the product specified time.

- Product is provided as sterile. Do not re-sterilize.
- With date sticker to identify the replacing date.
- If there is any breakage on plastic sleeve, please replace immediately.
- Do not use when package damaged or opened.
- This product is for disposable product, it is forbidden to reuse.
- Keep the product in a cool and dry place and avoid direct sunlight, high temperature and humidity.
- This product is used for clinical closed suction aspiration, used to absorb and exclude sputum from the patients airway. Do not use it for other purposes.
- Medical staff must be professionally trained to use this product.
- When inserting the suction catheter into the endotracheal tube, please select the tube specifications that consistent with the inner diameter of the endotracheal tube.
- The negative pressure suction recommended for adults is 100-120 mmhg. If the sputum is thick, the pressure can be increased appropriately, but it should not exceed 150 mmhg. The negative pressure suction recommended for infants and young children is 80-100 mmhg. The actual sputum suction time is recommended not to exceed 15 seconds.
- After sputum aspiration, the negative pressure switch should be locked to prevent accidental contact.
- When replacing the closed suction systems, first remove the breathing circuit, and then separate the artificial airway.
- It is strictly prohibited to use in case of packaging damage, pollution, product damage or beyond the expiry date.
- Use immediately after opening the package and destroy it after use.
- Patients allergic to PVC material should not use this product.

INSTRUCTION FOR USE:

- Tear the package of the product list, take the product out of the package, and pay attention to prevent contamination of suction catheter and accessories.
- Securely connect the machine end connector of the closed suction systems to the suction tube, and stick the expiry label in the package on the protective sleeve position of the suction catheter (expiry of 1 day for 24H type and 3 days for 72H type).
- Use the thumb and forefinger of one hand to move the body of the suction tube to the desired depth of the endotracheal tube; The model 72H closed suction systems must be operated by unscrewing the isolation valve.
- Rotate the top cover of the negative pressure switch of the closed suction systems 180 ° counterclockwise in the direction of rotation. When the "tick" sound is heard, the rotation will stop. At this time, the negative pressure switch pressing function is opened.

- When the suction catheter is inserted into the depth required for endotracheal intubation, press the negative pressure switch and slowly pull back the suction catheter to carry out sputum suction operation.
- When the suction tube is pulled back, when the suction tube's head end is pulled back beyond the isolation valve, the suction action is stopped and the suction ends. For model 72H, the isolation valve should be closed at this time.
- **Cleaning method of suction catheter:** A syringe or infusion set equipped with normal saline is connected with the irrigation port of the disposable closed suction systems, and the negative pressure switch is pressed while injecting normal saline for the flushing of the head after sputum suction; 5-10 ml of normal saline should be used. When flushing is completed, disconnect the connection end of flushing selection or close the water stop clip on the infusion set to stop conveying normal saline, and then disconnect the negative pressure switch button of suction catheter.
- After washing, rotate the upper cover of the negative pressure switch clockwise, stop the rotation when the "tick" sound is heard, and close the negative pressure switch pressing function to prevent unexpected pressing operation.
- Separate the machine end of the closed suction systems from the suction device, and complete the sputum suction operation.

Note: The method of adding humidifying fluid to the suction catheter:

When patients with sticky sputum or scabbed sputum need humidification fluid, humidification fluid can be injected through the MDI port.

If the product is packaged sterile;

- The device is sterilized with Ethylene Oxide. Sterile conditions are guaranteed only if the package is not damaged.

STERILEEO

- Do not re-sterilize. Re-sterilizing damages the product integrity.

INDICATIONS:

Closed Suction System is used for endotracheal/tracheostomy aspiration of adult and pediatric intubated patients.

CONTRAINDICATIONS:

Do not use a standard-length catheter in patients with tracheostomy. Insufficient length of the aspiration catheter can damage the mucous tissue.

TIME OF USAGE:

All of the Breathing Systems are intended for single use only. It is used during the application period.

SHELF LIFE:

For all Closed Suction Sets, Shelf Life is 3 years.

TARGET PATIENT GROUP:

Adult and pediatric patients.

USERS:

Users have the necessary expertise, training, and knowledge to use or install the medical device.

DISPOSAL METHOD:

Local regulations and/or hospital waste management procedures of the relevant country should be followed when disposing of the used products in order to avoid possible contamination.

PERFORMANCE DATA:

Air Resistance: The closed suction systems should be less than 0.03hPa/L/min for adults at 30L/min, and 0.07hPa/L/min for children at 15L/min.

Leakage: The closed suction systems should be less than 70mL/min for adults and 40mL/min for children at a pressure of 60cmH₂O according to the patient category.

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