RVENT

GAS SAMPLING LINE INSTRUCTION FOR USE **EN**

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GAS SAMPLING LINE INSTRUCTION FOR USE

INTENDED USE:

It is used to measure medical gases by connecting to a capnography device before transmitting to the patient. It helps to accurately measure CO₂, O₂ and N₂O levels. It prevents medical gases from being wasted. The product is provided as sterile and non-sterile.

WARNINGS:

- The medical device must only be used for the purpose specified under intended use.
- Do not use the medical device if the packaging is damaged.
- To avoid contamination and soiling, the product should remain packaged until ready to be used.
- Store in dry and cool place.
- Do not contain Latex.

If the product is packaged sterile;

- The device is sterilized with Ethylene Oxide. Sterile conditions are guaranteed only if the package is not damaged.

STERILE EO

- Do not re-sterilize. Re-sterilizing damages the product integrity.

INSTRUCTION FOR USE:

Attach the gas sampling line male luer lock connector to female luer connector on the patient end of the circuit. If swivel luer connector is present, adjust the elbow to desired position prior to securing. Attach other end (male or female) to the CO₂ monitor. Check for secure connections and proper function prior to use.

INDICATIONS:

It helps to accurately measure CO₂, O₂ and N₂O levels. It prevents medical gases from being wasted.

CONTRAINDICATIONS:

The device is used only as directed. Improper use of it will harm the patient.

TIME OF USAGE:

The device is intended for single use. It is used during the application period.

SHELF LIFE:

Shelf Life is 5 years.

TARGET GROUP:

Adult, pediatric and neonatal patients

USERS:

Users have the necessary expertise, education and knowledge to use or install the medical device.

DISPOSAL METHOD:

Local regulations and/or hospital waste management procedures of the relevant country should be followed when disposing of the used products.

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