RVENT

MASK INSTRUCTION FOR USE **EN**

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MASK INSTRUCTION FOR USE

INTENDED USE:

Anesthesia Mask: Disposable devices used for the transmitting of medical gases into the upper respiratory airways of the patients.

CPAP Mask: Disposable devices used for the transmitting of medical gases into the upper respiratory airways of patients who need CPAP therapy.

NIV Mask: The NIV mask is an accessory to a non-continuous ventilator. The mask is for single use patient use in home or in hospital/institutional environment. The mask is to be used by patients for whom CPAP or bi-level therapy has been prescribed.

WARNINGS:

General Warnings:

- The medical device is only to be used for the purpose specified under "Intended Use" section.
- The product is manufactured for single use only. Discard after use.
- The products must not be reused, reprocessed, or sterilized. Reuse, reprocessing, or sterilization may lead to a failure of the products and injury to the patient.
- To avoid contamination and soiling, the face mask should remain packaged until ready to be used.
- Do not place the face mask over open wounds that are prone to infection.
- If oxygen is used with the device, the oxygen flow must be turned off when the device is not operating.
- Do not place the face mask over open wounds that are prone to infection.
- If the patient experiences skin irritation, consult the physician.
- The mask cushion must expand correctly during inflation so that it fits hermetically onto the patient's facial contours. A medium pressure in the mask cushion ensures the best fit and seal.
- Excessive inflation of the mask cushion may reduce the useful surface and can lead to obstruction of the airways.

- The patient's physician should be contacted if the patient experiences the following symptoms while using the mask or after removing it: Unusual chest discomfort, shortness of breath, stomach distension, belching or severe headache; drying of the eyes, eye pain or eye infections; blurred vision.
- The patient should be constantly monitored during the use of this device.
- It should not be wetted, rinsed or washed.
- It must be stored in cool and dry places.

Additional Warnings for Anesthesia Masks:

- Irritation of the airways may occur in the case of lengthy CPAP application using scented masks.
- CPAP application: This face mask does not include an anti-asphyxia valve via which the patient could breathe independently in the case of failure of the ventilator.
- When using a headgear (optional), this must be attached via the hook ring on the face mask.

Additional Warnings for NIV Masks:

- This mask is designed for use with CPAP or bi-level systems recommended by your health professionals. Do not wear this mask unless the CPAP or bi-level system is turned on and operating properly. Do not block or try to seal the exhalation port.
- When the CPAP machine is not operating, enough fresh air will not be provided through the mask and exhaled air may be re-breathed. This warning applies to most models of CPAP systems.
- At low CPAP or EPAP pressures, the flow through the exhalation port may be inadequate to clear all exhaled gas from the tubing. Some re-breathing may occur.
- This mask is not recommended for patients who are taking any prescription of drug that may cause vomiting.
- When including an additional exhalation device to the patient circuit, you may need to adjust pressure level to compensate for the additional leak of the exhalation device.

INSTRUCTION FOR USE:

Anesthesia Masks: Check the cushion for adequate inflation by gently pressing it with hand, if necessary, adjust mask cushion via the disposable syringe connected to the check valve. Place mask cushion on the patient's chin first, roll cushion up the face with gentle but firm pressure. The Hook ring can be used in conjunction with a headstrap or removed when not needed. Connect the mask to the gas source such as Manual Resuscitator, Breathing Circuit etc.

CPAP Masks: Hold the Masks Base in one hand, expand the Harness with your free hand just enough to fit comfortably over your head. Attach the upper strap to the ring hook of the mask. Attach the upper strap to the ring hook of the mask. Lightly hold the mask against your face and spread the Harness over your head. Adjust the top horizontal straps to ensure the mask sits comfortably and securely on the nose. Adjust the top crown straps as needed. Connect the CPAP machine hose running at prescribed pressure to the mask. If necessary, attach the oxygen supply or pressure sensing device to the standard 4.3mm port on the mask. If necessary, attach the adjustable PEEP Valve to the expiration port on the mask and adjust PEEP Valve.

NIV Masks: Open the headgear to a large size setting. Hold the mask and place the cushion against the face by first setting the base of the cushion under the lower lip then roll the cushion up to nose and the Forehead Pad lightly resting on your forehead. Adjust the upper straps to a comfortable position by using the tabs on both sides of the headgear. Do not pull the straps tight. The cushion should lightly touch your face and the upper straps will rest above your ears. Adjust the lower straps to a comfortable position by using the tabs on both sides of the headgear. Do not pull the straps tight. The lower straps will rest below your ears and the headgear will cross at the base of your head. Connect the tubing (included with the CPAP or bi-level device) to the tubing swivel. Turn on the airflow. Lie down and breathe normally. Adjust the headgear evenly to reduce leaks or pressure points at the bridge of the nose. After usage, remove the mask by grabbing the back headgear strap and sliding the entire assembly forward over your head.

INDICATIONS:

Anesthesia Masks: It provides oxygen support and assist ventilation in patients with impaired respiratory function.

CPAP Masks: The mask is to be used by patients for whom CPAP or bi-level therapy has been prescribed.

NIV Masks: The mask is to be used by patients for whom CPAP or bi-level therapy has been prescribed.

CONTRAINDICATIONS:

Anesthesia Masks: Do not use the masks on patients who are uncooperative, obtunded, unresponsive or unable to remove the mask. Do not place the masks over open wounds that are prone to infection.

CPAP Masks: Excessive gastroesophageal reflux, impaired cough reflex, Hiatal hernia, Impaired cardiac sphincter. Also, do not use the masks on patients who are uncooperative, obtunded, unresponsive or unable to remove the mask.

NIV Masks: Do not use the masks on patients who are uncooperative, obtunded, unresponsive or unable to remove the mask.

TIME OF USAGE:

All of the breathing masks are intended for single use only. It is used during the application period.

SHELF LIFE:

For all of the breathing masks, Shelf Life is 3 years.

TARGET PATIENT GROUP:

Anesthesia Masks: Adult, pediatric and neonatal patients

CPAP Masks: Adult and pediatric patients

NIV Masks: Adult and pediatric patients

USERS:

Users have the necessary expertise, training and knowledge to use or install the medical device.

CHOKING HAZARD:

Keep out of reach of children. Because, the mask contains small parts.

DISPOSAL METHOD:

Local regulations and/or hospital waste management procedures of the relevant country should be followed when disposing of the waste products.

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